

APESIN multi

QUICK & EASY



Foaming disinfectant multi-purpose cleaner

- Versatile ■ Safe ■ Highly efficient

Product Profile

- Disinfectant multi-purpose cleaner with a broad activity against bacteria, yeast, viruses (limited spectrum) and fungi in medical, institutional, and public areas.
- APESIN multi Quick & Easy ensures effective disinfecting and cleaning results in one step.
- The highly concentrated formulation minimizes transport weight, and leads to a lower inventory.
- The Quick & Easy portable dosing system offers accurate and safe dosage.
- Free of dye and perfume.
- By the abandonment of ingredients such as aldehyde, phenol, alcohol, and chlorine there is no odour nuisance.
- No aerosol thanks to innovative foam solution and therefore perfectly suitable in large-scale areas.
- Being CLP-free in application, it offers maximum user-safety.
- Skin tolerance has been dermatologically tested.

Area of application

- APESIN multi Quick & Easy is suitable for hard surfaces.
- Versatile, can be used on all water-resistant surfaces in institutional areas, e.g. public facilities, offices, gyms.
- Also suitable in nursing homes, doctor's surgeries, and old people's home.
- In sanitary areas where limescale-dissolving power is needed, use e.g. SANET daily Quick & Easy which is particularly suitable for removal of lime soap and leaves pleasant lemon scent after application.
- High compatibility on common plastic materials, e.g. PMMA/Polymethyl methacrylate Plexiglas 8N, POM/Polyoxymethylene Hostaform type C 9021, type 13021.
- Excellent compatibility on electroplated surfaces, such as chrome, stainless steel, and brass, as well as on glass.

Application note

- Use separate trigger per APESIN product and use sticker for differentiation.
- Apply foam onto a cloth and completely wet the surface.
- Moisten the cloth sufficiently with APESIN multi Quick & Easy in order to guarantee appropriate wetting of the surface.
- Allow the solution to work for at least 1 minute.
- We recommend to empty the water bottle after a working day, and to refill the bottle with fresh water before next use.
- For detailed information see next page: Standards & Spectrum of activity.

Ingredient declaration

Ingredients per 100g: 2,45g DIDECYLDIMONIUM CHLORIDE, <5% Non-ionic surfactants

Sustainable development and production




Product safety, storage and environmental instructions

Safety: For professional use only. For further information, see Safety Data Sheet. Use biocides safely. Always read the label and product information before use.


Storage: Store at room temperature in the original container. Keep from freezing.

Environmental: Only give completely emptied packaging to special waste disposer.


Application and Dosing




Use separate trigger for APESIN multi Quick & Easy.




Dose in relation to application. Follow below instructions.




Empty water bottle after a working day.

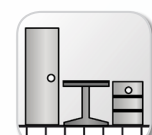


Surface disinfection/cleaning: Apply foam onto a cloth and completely moisten the surface.



Specific disinfection: suitable for areas with specific hygiene requirements.



Sanitary disinfection: Apply foam onto a cloth and completely moisten the surface.



Sanitary cleaning: For increased limescale dissolving power use e.g. SANET daily Quick & Easy before disinfecting.

predefined dosing

Sales units:
Order No.: 714319 6 x 325 ml

Your trusted partner on site:

pH-Value  11

APESIN multi

**QUICK
& EASY**


Foaming disinfectant multi-purpose cleaner

Disinfection activity of APESIN multi Quick & Easy

		exposure time
bactericidal		
EN 1276	high organic load	5 min.
EN 13727	high organic load	1 min.
EN 13697 I&I	high organic load	5 min.
EN 16615	high organic load	1 min.
yeastocidal		
EN 1650	high organic load	5 min.
EN 13624	high organic load	1 min.
EN 13697 I&I	high organic load	15 min.
EN 16615	high organic load	1 min.
fungicidal		
EN 13624	high organic load	30 min.
EN 16615	high organic load	30 min.
virucidal		
EN 14476 effective against enveloped viruses (incl. HIV, HBV, HCV)	high organic load	1 min.
EN 14476 limited spectrum virucidal activity	high organic load	30 min.
EN 14476 Vacciniavirus	high organic load	1 min.
En 14476 Adenovirus	high organic load	30 min.
EN 14476 Norovirus	high organic load	5 min.
EN 14476 Rotavirus	without organic load	1 min.

		exposure time
VAH-listed - surface disinfection in medical areas bactericidal and yeastocidal activity	with mechanical action dirty conditions	5 min.

Quick & Easy system automatically ensures a correct dilution of min. 10% (v/v) from the product concentrate.

